TERAFLOAT®

POWER DENTISTRY 2020



Terafloat ® Designed and manufactured by Swissvet Veterinary Products



© SWISSVET 2020

2218 N Central St Knoxville, TN 37917 USA

WWW.TERAFLOAT.COM +1 865 540 8830

THE TERAFLOAT ®

LIGHTWEIGHT PERFECTION

The concept of the Terafloat is simple: a lightweight instrument with lithium power and a range of interchangeable shafts with unique features. The combination of modern materials, a clever design and Swiss quality makes the Terafloat the perfect instrument for state-of-the-art equine dentistry treatment in the 21st century.

The Terafloat is sold since 2015 and has been a tremendous success among equine veterinarians and it is equally liked by horse owner and trainers.

FOR VETS FROM A VET

The Terafloat was developed by the board certified Swiss veterinarian Dr. Ruedi Steiger, DACVIM. He moved to the US for a residency program in equine medicine but found his true passion in equine dentistry. Dr. Steiger owns Swissvet and he is practicing and teaching equine dentistry.

Dr. Steiger combined 20+ years of dentistry experience with Swiss engineering to design his version of a perfect equine Powerfloat. The high precision parts for the Terafloat are machined in Switzerland exclusively for Swissvet.





ONE NAME - THREE LINES: ECO, LED AND FLX

Terafloat offers three lines of power instruments: The ECO line, the LED line and the FLX line. Each line has a wide range of instrument sets from basic to professional, to satisfy every budget and every treatment approach. Together they meet the demands from the general large animal practitioner to the equine dentistry specialist.

Assembly and service is performed in several locations around the world. The Terafloat is distributed in over 20 countries. Swiss quality and a one year warranty are standard.

HOW TO CHOOSE A TERAFLOAT

- 1. Select a model line: ECO, LED or FLX
- 2. Pick the routine shafts: Disc shaft and/or Apple core shaft
- 3. Consider elite shafts: Pony and/or Incisor shaft
- 4. Find the set that meets your selection



TERAFLOAT ECO STURDY AND AFFORDABLE The Terafloat ECO motor features: Variable speed trigger • Optional trigger locking mode Electronic shut-off protection • Lithium batteries 3Ah • Battery charge indicator lights Removable shafts TERAFLOAT ECO WITH ROUTINE DISK SHAFT INTERCHANGEABLE HEADS STITLE SHAFTS DIAMOND DISCS AND BURRS SHAFTS AV LOW PROFILE BURR ROTATING HEAD APPLE CORE SHAFT PONY SHAFT INCISOR SHAFT TURNFLEX SHAFT GUARDED GRINDING OR CUTTING WHEEL

WATER RESISTANT - MAINTENANCE FREE - SWISS MADE

TERAFLOAT

The Terafloat ECO line is amazing for everyday dentistry. It offers all Terafloat features but without the LED illumination. It is a sturdy and versatile solution for a great price.

The two routine shafts have the Turnflex mechanism that allows switching from a straight or curved position allowing the perfect positioning of the grinding head. The apple core shaft and the disk shaft are identical, but use a different grinding head. The diamond coated Terafloat burrs and disks are Swiss made and they are interchangeable. The routine shafts are efficient and produce precise dental work.

The two elite shafts, the pony shaft and the incisor shaft, use the same motor and they are easily interchanged. The pony shaft has a low profile guarded burr that reaches tight spaces and the incisor shaft with a large cutting or grinding wheel allows for efficient and precise incisor work.

The ECO line is great for the general practitioner that wants a modern instrument with smart features that allows for detailed equine dentistry procedures at an affordable price. When upgrading to an advanced or professional set, all corners of dentistry are covered with the ECO line.

Several Terafloat Eco sets are available. The components are modular and interchangeable, making it easy to upgrade from one set to the next.

T-0036G: TERAFLOAT ECO BASIC SET

(1) Motor ECO, (2) Batteries 3.0Ah, (1) Charger, (1) Routine ECO shaft with guarded rotary disc T3-C10, (1) Tool Bag, (1) Long case. Also available with Head with non-guarded rotary disc T2-C10 (T-0036).

T-0039G: TERAFLOAT ECO BASIC PLUS SET

(1) Motor ECO, (2) Batteries 3.0Ah, (1) Charger, (1) Routine ECO shaft with guarded rotary disc T3-C10, (1) Head with non-guarded Apple Core Burr A2-M8, (1) Tool Bag, (1) Long case. Also available with Head with non-guarded rotary disc T2-C10 (T-0039).

T-0042G: TERAFLOAT ECO STANDARD SET

(2) Motors ECO, (3) Batteries 3.0Ah, (1) Charger, (1) Routine ECO shaft with guarded rotary disc T3-C10, (1) Routine ECO shaft with non-guarded Apple Core Burr A2-M8, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0042).

T-0060G: TERAFLOAT ECO ADVANCED SET (INCISOR)

(2) Motors ECO, (3) Batteries 3.0Ah, (1) Charger, (1) Routine ECO shaft with guarded rotary disc T3-C10, (1) Routine ECO shaft with non-guarded Apple Core Burr A2-M8, (1) Incisor shaft, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0060).

T-0001G: TERAFLOAT ECO ADVANCED SET (PONY)

(2) Motors ECO, (3) Batteries 3.0Ah, (1) Charger, (1) Routine ECO shaft with guarded rotary disc T3-C10, (1) Routine ECO shaft with non-guarded Apple Core Burr A2-M8, (1) Pony shaft, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0001).

T-0027G: TERAFLOAT ECO PROFESSIONAL XS SET

(2) Motors ECO, (3) Batteries 3.0Ah, (1) Charger, (1) Routine ECO shaft with guarded rotary disc T3-C10, (1) Routine ECO shaft with non-guarded Apple Core Burr A2-M8, (1) Incisor shaft, (1) Pony shaft, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0027).

T-0063G: TERAFLOAT ECO PROFESSIONAL XL SET

(4) Motors ECO, (6) Batteries 3.0Ah, (1) 4 Bay-charger, (1) Routine ECO shaft with guarded rotary disc T3-C10, (1) Routine ECO shaft with non-guarded Apple Core Burr A2-M8, (1) Incisor shaft, (1) Pony shaft, (1) Incisor Speculum, (1) Tool Bag, (1) Large case. Also available with a head with non-guarded rotary disc T2-C10 (T-0063).


















TERAFLOAT LED

LIGHT WEIGHT PERFECTION

The LED motor features:

- Variable speed trigger
- Optional trigger locking mode
- Electronic shut-off protection
- Lithium batteries 3Ah
- Battery charge indicator lights
- Removable shafts
- Wired for LED shaft

TERAFLOAT LED WITH ROUTINE DISC SHAFT

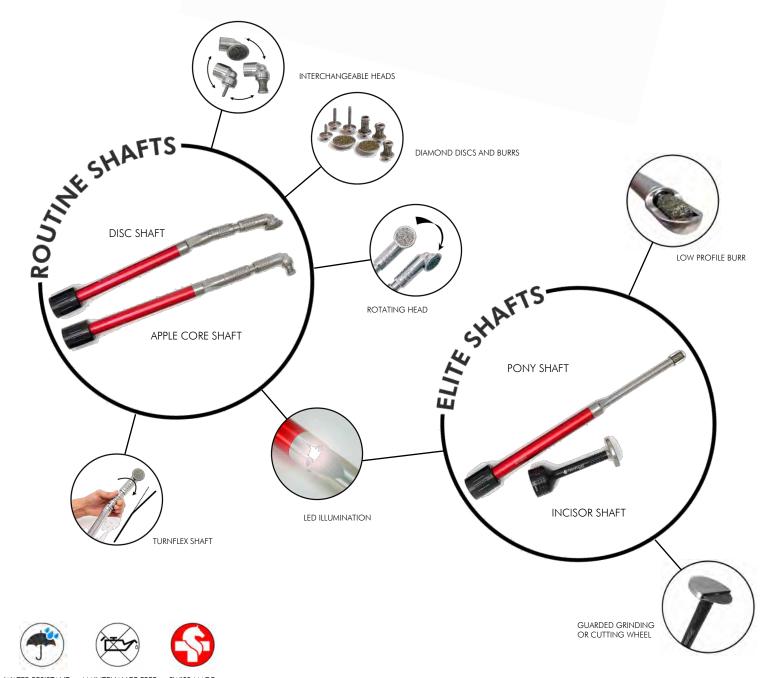

TERAFLOAT

The Terafloat LED line is the perfect power float for equine dentistry. It is lightweight, battery powered, water proof, maintenance free, durable and the built-in LED lights illuminate the oral cavity for perfect examination and treatment.

The two routine shafts have the Turnflex mechanism that allows switching from a straight or curved position allowing the perfect positioning of the grinding head. The apple core shaft and the disk shaft are identical, but use a different grinding head. The diamond coated Terafloat burrs and disks are Swiss made and they are interchangeable. The routine shafts are efficient and produce precise dental work.

The two elite shafts, the pony shaft and the incisor shaft, use the same motor and they are easily interchanged. The pony shaft has a low profile guarded burr that reaches tight spaces and the incisor shaft with a large cutting or grinding wheel allows for efficient and precise incisor work.

The Terafloat LED is an efficient and practical instrument with smart features that allows proficient equine dentistry procedures of the 21st century. Several Terafloat LED sets are available to satisfy all dentistry needs. The modular components are interchangeable, making it easy to upgrade from one set to the next.

T-0004G: TERAFLOAT LED BASIC SET

(1) Motor LED, (2) Batteries 3.0Ah, (1) Charger, (1) Routine LED shaft with guarded rotary disc T3-C10, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 T-0004).

T-0007G: TERAFLOAT LED BASIC PLUS SET

(1) Motor LED, (2) Batteries 3.0Ah, (1) Charger, (1) Routine LED shaft with guarded rotary disc T3-C10, (1) Head with non-guarded Apple Core Burr A2-M8, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0007).

T-0014G: TERAFLOAT LED STANDARD SET

MOST POPULAR

Motors LED, (3) Batteries 3.0Ah, (1) Charger, (1) Routine LED shaft with guarded rotary disc T3-C10, (1) Routine LED shaft with non-guarded Apple Core Burr A2-M8, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0014).

T-0050G: TERAFLOAT LED ADVANCED SET (INCISOR)

(2) Motors LED, (3) Batteries 3.0Ah, (1) Charger, (1) Routine LED shaft with guarded rotary disc T3-C10, (1) Routine LED shaft with non-guarded Apple Core Burr A2-M8, (1) Incisor shaft, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0050).

T-0053G: TERAFLOAT LED ADVANCED SET (PONY)

(2) Motors LED, (3) Batteries 3.0Ah, (1) Charger, (1) Routine LED shaft with guarded rotary disc T3-C10, (1) Routine LED shaft with non-guarded Apple Core Burr A2-M8, (1) Pony shaft, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0053).

T-0011G: TERAFLOAT LED PROFESSIONAL XS SET

(2) Motors LED, (3) Batteries 3.0Ah, (1) Charger, (1) Routine LED shaft with guarded rotary disc T3-C10, (1) Routine LED shaft with non-guarded Apple Core Burr A2-M8, (1) Incisor shaft, (1) Pony shaft, (1) Tool Bag, (1) Long case. Also available with a head with non-guarded rotary disc T2-C10 (T-0011).

T-0056G: TERAFLOAT LED PROFESSIONAL XL SET

(4) Motors LED, (6) Batteries 3.0Ah, (1) 4 Bay-charger, (1) Routine LED shaft with guarded rotary disc T3-C10, (1) Routine LED shaft with non-guarded Apple Core Burr A2-M8, (1) Incisor shaft, (1) Pony shaft, (1) Incisor Speculum, (1) Tool Bag, (1) Large case. Also available with a head with non-guarded rotary disc T2-C10 (T-0056).






TERAFLOAT FLX

FLEXIBLE PERFECTION

The FLX motor features:

- Variable speed trigger with trigger lock
- Electronic shut-off protection
- Lithium batteries 6Ah
- Battery charge indicator lights
- Belt system with shaft holster
- Reinforced flexible drive shaft
- Waterproof magnetic shaft connection



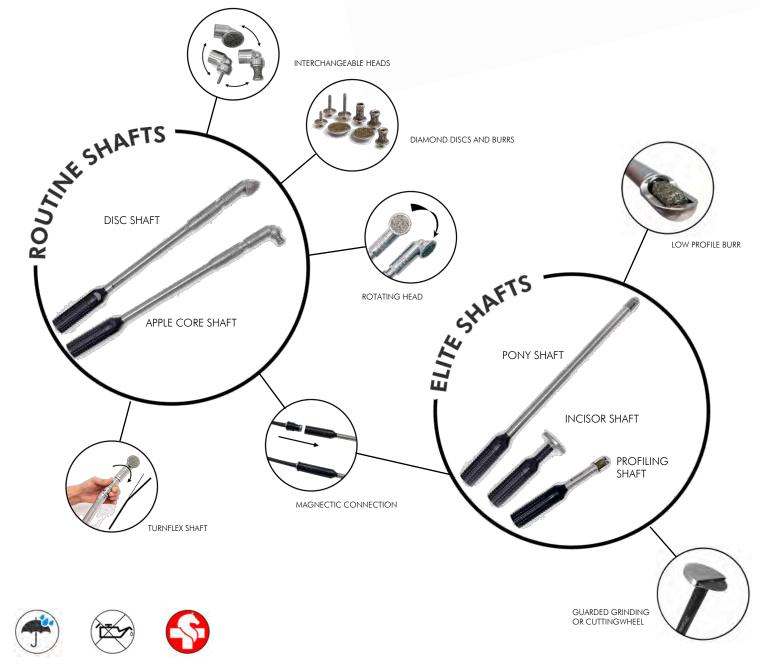

WATER RESISTANT - MAINTENANCE FREE - SWISS MADE

TERAFLOA

The Terafloat FLX line is perfect for the one handed treatment technique often favored in Europe. The motor pivots on a carrying belt with an instrument holster to hold the grinding shaft if needed. The connection between the reinforced flexible drive shaft and the grinding shafts is magnetic and waterproof.

The two routine shafts have the Turnflex mechanism that allows switching from a straight or curved position allowing the perfect positioning of the grinding head. The apple core shaft and the disk shaft are identical, but use a different grinding head. The diamond coated Terafloat burrs and disks are Swiss made and they are interchangeable. The routine shafts are efficient and produce precise dental work.

The three elite shafts, the pony, the incisor and the profiling shaft. The pony shaft has a low profile guarded burr that reaches tight spaces and the incisor shaft with a large cutting or grinding wheel allows for efficient and precise incisor work. The profiling hand piece handles detailed work in the rostral aspect of the oral cavity including incisor work and '06 profiling.

Various Terafloat FLX sets are available, and the components are modular and interchangeable, making it easy to upgrade from one set to next level.

T-0077G: TERAFLOAT FLX BASIC SET

(1) Motor Eco, (1) Belt with Holster, (2) Batteries 6.0Ah, (1) Charger, (1) Flexible Drive Shaft, (1) Terafloat FLX Shaft with guarded rotary disc T3-C10, (1) Tool Bag, (1) Long Case. Also available with a head with non-guarded rotary disc T2-C10 (T-0024).

T-0080G: TERAFLOAT FLX BASIC PLUS SET

(1) Motor Eco, (1) Belt with Holster, (2) Batteries 6.0Ah, (1) Charger, (1) Flexible Drive Shaft, (1) Terafloat FLX shaft with guarded rotary disc T3-C10, (1) Head with non-guarded Apple Core Burr A2-M8, (1) Tool Bag, (1) Long Case. Also available with a head with non-guarded rotary disc T2-C10 T-0080)

T-0083G: TERAFLOAT FLX STANDARD SET

(1) Motor Eco, (1) Belt with Handpiece Holster, (2) Batteries 6.0Ah, (1) Charger, (1) Flexible Drive Shaft, (1) Terafloat FLX shaft with guarded Rotary disc T3-C10, (1) Terafloat FLX shaft with non-guarded Apple Core Burr A2-M8, (1) Tool Bag, (1) Long Case. Also available with a head with non-guarded rotary disc T2-C10 (T-0083)

T-0089G: TERAFLOAT FLX ADVANCED SET (INCISOR)

(1) Motor Eco, (1) Belt with shaft Holster, (2) Batteries 6.0Ah, (1) Charger, (1) Flexible Drive Shaft, (1) Terafloat FLX shaft with guarded rotary disc T3-C10, (1) Terafloat FLX shaft with non-guarded Apple Core Burr A2-M8, (1) Terafloat FLX Incisor handpiece, (1) Tool Bag, (1) Long Case. Also available with a head with non-guarded rotary disc T2-C10 (T-0089)

T-0096G: TERAFLOAT FLX ADVANCED SET (PONY)

(1) Motor Eco, (1) Belt with shaft Holster, (2) Batteries 6.0Ah, (1) Charger, (1) Flexible Drive Shaft, (1) Terafloat FLX shaft with guarded rotary disc T3-C10, (1) Terafloat FLX shaft with non-guarded Apple Core Burr A2-M8, (1) Terafloat FLX Pony handpiece, (1) Tool Bag, (1) Long Case. Also available with a head with non-guarded rotary disc T2-C10 (T-0096)

T-0092G: TERAFLOAT FLX PROFESSIONAL XS SET

(1) Motor Eco, (1) Belt with shaft Holster, (2) Batteries 6.0Ah, (1) Charger, (1) Flexible Drive Shaft, (1) Spare Drive Cable with changing keys, (1) Terafloat FLX shaft with guarded rotary disc T3-C10, (1) Terafloat FLX shaft with non-guarded Apple Core Burr A2-M8, (1) Terafloat FLX Incisor handpiece, (1) Terafloat FLX Pony handpiece, (1) Tool Bag, (1) Long Case. Also available with a head with non-guarded rotary disc T2-C10 (T-0092)

T-0109G: TERAFLOAT FLX PROFESSIONAL XL SET

(1) Motor Eco, (1) Belt with shaft Holster, (2) Batteries 6.0Ah, (1) Charger, (1) Flexible Drive Shaft, (1) Spare Drive Cable with changing keys, (1) Terafloat FLX shaft with guarded rotary disc T3-C10, (1) Terafloat FLX shaft with non-guarded Apple Core Burr A2-M8, (1) Terafloat FLX Incisor handpiece, (1) Terafloat FLX Pony handpiece, (1) Terafloat FLX Profiling handpiece, (1) Tool Bag, (1) Long Case. Also available with a head with non-guarded rotary disc T2-C10 (T-0109)





TERAFLOAT COMPONENTS

Rotary Disc T2-C10 (T-0202) Coarse diamond coated grinding disc for non-guarded head (T-0180)



Rotary Disc T3-C10 (T-0206) Coarse diamond coated grinding disc for guarded head (T-0181)

Apple Core Burr A2-M8 (T-0212) Diamond coated apple core grinding burr for non-guarded head

Apple Core Burr Canine A4-M8 (T-0217) Diamond coated apple core grinding burr for nonguarded head with additional inside coating for canine work

Cylindrical Burr Large Z1-M8 (T-0217) Diamond coated large cylindrical grinding burr for non-guarded head.


Diamond coated diastema burrs for non-guarded head only.

Conical Short D1-F4 (T-0261) Full coating. Cylindrical Long D2-F4 (T-0262) Full coating. Cylindrical Extra Long D3-F4 (T-0263) Tip not coated.

Incisor Wheel S1-F4 (T-0285) Diamond coated grinding wheel for incisor shaft.

Pony Shaft Burr P1-C8 (T-0283) Diamond coated grinding burr for pony shaft or profiling shaft (FLX).

Terafloat ECO (or LED) Incisor Shaft (T-0159) Terafloat incisor shaft with guarded diamond wheel for incisor work, works with LED and ECO motor

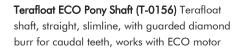





Terafloat FLX Pony Shaft (T-0143) FLX pony shaft, straight, slimline, with guarded diamond burr for caudal teeth

Terafloat FLX Profiling Shaft (T-0146) FLX short profiling shaft with diamond coated burr, with or without protection

Terafloat FLX drive shaft cable (T-0173) Replacement cable 6mm for FLX flexible drive shaft

Clinic Charger (T-0128) Milwaukee 4-bay charger (used in professional sets)



























USER INSTRUCTIONS

IMPORTANT

Always use sufficient sedation and a good quality full mouth speculum for treatment. Perform the procedures only if you have adequate training and experience with equine dentistry. Remove dental material conservatively - avoid that horses receive excessive treatment. If you have any questions or concerns please do not hesitate to contact Swissfloat for assistance.

These instructions are valid for all three product lines of the Terafloat, ECO, LED and FLX.

Incisors (1) and Canines (also see 9): This procedure is best done at the beginning of the dental work when sedation is the strongest, with the Swissvet incisor speculum. Use the grinding disc for work on incisors (1) and additionally the apple core burr for smoothening the canines. Advanced users may consider the Terafloat Incisor float (9).

Points on lower (2) and upper arcade (3): Use the apple core burr and place it at a 45 deg angle on the lingual (2, lower arcade) or vestibular (3, upper arcade) side of the cheek teeth where points need to be smoothened and move caudally. Closing the speculum by one click on the treatment side can increase the space between cheek and teeth. Remove sharp points conservatively.













Hooks on lower (4) and upper arcade (5): Hooks can be reduced with grinding disks, the apple core burrs or large cylindrical burrs, depending on the pathology as well as the space between upper and lower arcade.

When floating caudal hooks you may flex the shaft toward the lingual side to push the tongue away and hold the float straight to work your way from the lingual side onto the hooks (4). You may use your hand to guide the grinding head for rostral hooks (1/206). Flexing the shaft down helps avoiding the incisor plates of the speculum (5).

Waves and steps: Use the grinding disc (see 4) or, if space is very limited, the large cylindrical burr (6) to reduce waves or steps. Do not accidentally open a pulp cavity and don't take more than two cheek teeth out of occlusion one either side to avoid exæssive stress on the remaining teeth in occlusion.

Bit seat (7): Use the apple core burr to place a bit seat, maybe guiding the instrument with your second hand for better control of the procedure. Make sure not to open the pulp with this procedure.

Diastema (8): Use the long mandibular (D2), long maxillary (D3) or short (D1) diastema burr to enlarge an interdental space with precaution. Make sure you understand the risk of this procedure and perform it only if other treatments have failed.

Incisor work (9) using the incisor shaft: If not using the flat diamond disk: use the incisor shaft. Attention: this hand piece grinds very aggressively; make sure you don't open a pulp. Heat production is typically not a concern but check manually for heat. Use this hand piece to grind (9) or to cut the incisor to the desired length or shape. It is best to block the trigger lock and hold the motor/shaft with one hand and the mandible or maxilla with the other.

Working with small breeds or in tight spaces (10) using the pony shaft: The pony shaft offers the perfect solution if the anatomy doesn't allow enough room for routine shafts. The pony shaft is available for the ECO, FLX and the LED line. It has a small profile diamond burr to grind the tooth into the desired shape or to shorten it enough to apply the disk shaft or the apple core shaft to finish the job.













DENTISTRY ACCESSORIES -

Swissvet Standard Speculum (S-0012) McPherson style speculum, 5 adjustment levels, stainless steel (magnetic) with Biothane straps, standard AND half-circle plates

Swissvet Optimum Speculum (S-0015) Millennium style speculum, 22 adjustment levels, stainless steel (magnetic) with Biothane straps, standard AND half-circle plates

Standard Plates (S-0050) Stainless steel (magnetic) standard plates for full-mouth speculum

Half Circle Plates (S-0054) Stainless steel (magnetic) half-circle plates for full-mouth speculum

Straight Plates (S-0058) Straight speculum plates for incisor work (stainless steel with plastic protection)

Endodontic Plates (S-0060) Stainless steel (magnetic) standard plates for full-mouth speculum with gap for endodontic treatment (pair)

TARO speculum (S-0016) Aluminum full-mouth speculum with rapid-close knobs, adjustable incisor plates, and Biothane straps (made in Germany)

Swissvet Mighty Speculum (S-0023) Standardsized stainless steel speculum, 10 adjustment levels, with Biothane straps and half-circle plates (made in USA)

Swissvet Silverslide Speculum (S-0024)

Heavy-duty stainless steel speculum, infinitely adjustable, with Biothane straps and half-circle plates (made in USA)

Modified Gunther Speculum (S-0022) Modified

Gunther speculum with moveable bite plates for

MITSE technique; stainless steel with Biothane

strap







Pony Mighty Speculum (S-0019) Pony-sized stainless steel speculum, Mighty style, with Biothane straps and half-circle plates (made in Pakistan)





Swissvet Incisor Speculum (S-0026) Incisor speculum with adjustable elastic strap



port ring with pivoting base, rope with cleat






Spreader Bar (S-0030) Stainless steel spreader bar for full-mouth speculums (fits McPherson or Millennium type speculums)



Cheek Retractor (S-0031) Stainless steel cheek retractor for full-mouth speculums

Swissvet Twitch (S-0115) L-shaped stainless steel twitch with 6mm rope, single person model with brass bolt snap

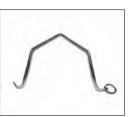







DENTISTRY ACCESSORIES

Headstand Standard (S-0035) Hoofjack head stand, standard model with a red base



Headstand Articulated (S-0036) Hoofjack head stand with advanced articulated cradle and a red base

Instrument Rack (S-0037) Instrument rack for Hoofjack head stand

Flush Syringe 400mL (S-0110) Flush syringe 400 mL, two-handed model, with spare seal



Wolf Tooth Set Small (S-0201) Wolf tooth extraction set with 3 elevators, 2 forceps, and pouch

Periodontometer Set (S-0250) Erbrich periodon-

tometer set with stainless steel handle and 2 periodontometer screw on tips, long (90deg) and

Periodontometer tip LONG (S-0252) Stainless

Periodontometer tip SHORT (S-0253) Stainless

Tarter Removal Scaler (S-0405) Scaler with flat

and pointy tip, for canine tooth tartar removal.

short (60deg), made in Germany.

steel replacement tip long 90 deg.

steel replacement tip short 60 deg.



Flush Syringe 300mL (S-0111) Flush syringe with pistol grip 300 mL, one-handed model, with spare seal

Dental Mirror 50mm (S-0040) Dental mirror 50mm, stainless with T-handle. Replacement mirror available upon request (S-0041)

Illuminated Dental Mirror 48mm (S-0042) Illuminated dental mirror 48mm with LED light, one AA battery, and protective pouch (made in Germany). Replacement mirror available upon request (S-0044)

Examination Set (S-0400) Examination and exploration set with 2 handles, straight (S-0403) and Tbar (S-0402), 12 screw on tips (S-0401), and pouch







Wolf Tooth Elevators Long Pair (S-0204) Set of 2 stainless steel elevators 30 cm length




Wolf Tooth Forceps Angled 11" (S-0255) Stainless steel 11" wolf tooth forceps, angled.

Wolf Tooth Forceps Straight 11" (S-0256) Stainless steel 11" wolf tooth forceps, straight.







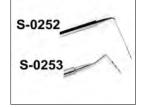









DENTISTRY ACCESSORIES -

S-Float (S-0101) S-shaped hand float with carbide chips (made in Germany)



Π

Hippodent Handle Short (S-0240) or Long

(S-0241) Short hippodent hand float handle 10" with pistol grip OR long hippodent hand float handle 16" with pistol grip: without blade (made in Germany)

Hippodent Blade Straight (S-0242) Straight blade with carbide chips, fits short or long hippodent handle (made in Germany)

Hippodent Blade Curved Up (S-0243) Curved up blade with carbide chips, for caudal work, fits short or long hippodent handle (made in Germany)

Hippodent Blade Curved Down (S-0244) Curved down blade with carbide chips, for bit seat, fits short or long hippodent handle (made in Germany)

Incisor Extraction Set Swissvet (S-0301) Incisor forceps 11" (S-0302), mallet (S-0305), fine elevator (S-0306), flat elevator (S-0308), and root elevator set (S-0420)

Root Elevator Set (S-0420) Set of 4 stainless steel

elevators with large brown handle

Swissvet Mallet (S-0305)

Stainless Steel Mallet, 750g

blade elevator for gingiva separation

Elevator Flat (S-0308) Stainless steel flat curved

Elevator Fine (S-0306) Stainless steel elevator with fine tip for fragment removal


Incisor Forceps 11" (S-0302) Stainless steel forceps with 11" handle





Octagonal Elevator Set (S-0419) Gingiva and molar fragment elevator set with 2 t-bar handles and 10 inserts (5 flat and 5 convex), stainless



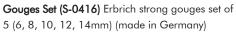

Scaler Filling Instrument (S-0417) Erbrich filling instrument for diastema treatment, with scaler end and 30mm long ball point 2.8mm, made in

Reynolds Caps removal forceps set (S-0220) Set of 2 forceps for deciduous cheek tooth removal, stainless steel.







Gingiva Elevator Set (S-0418) Set of 2 stainless steel T-bar handle elevators for gingiva or fragments







Germany.



DENTISTRY ACCESSORIES

Molar Extraction Set (S-0225) Stainless steel extraction forceps set with (1) 3-prong left, (1) 3prong right, (1) 4-prong, (1) molar spreader, and transport pouch

Molar Extraction Forceps 3P Left (S-0226) Stain-

less steel 3-prong (one left) forceps with 19"

round handle

Molar Extraction Forceps Concave (S-0323) Universal molar forceps, concave jaws, round handle





Reverse Molar Extraction Forceps Lower (S-0239) Stainless steel straight reverse molar extraction forceps for mandibular 06 with 19" round handle


Reverse Molar Extraction Forceps Upper (S-0239)


Stainless steel curved reverse molar extraction

forceps for maxillary 06 with 19" round handle





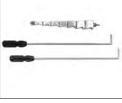

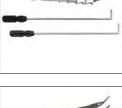









Molar Extraction Forceps 3P Right (S-0227) Stainless steel 3-prong (one right) forceps with 19" round handle

Molar Extraction Forceps 4P (S-0228) Stainless

Molar Extraction Forceps 4P Flat (S-0237) Stain-

Molar Extraction Forceps 4P Pony (S-0233) Stain-

Molar Forceps Universal Long (S-0229) Stainless steel forceps with flat serrated jaw and 19" round

less steel pony sized 4-prong forceps with 19"

round handle

handle

less steel 4-prong forceps with 19" flat handle

steel 4-prong forceps with 19" round handle

Fulcrum With Aluminum Blocks (S-0236) Fulcrum with two stainless steel handles and 7 aluminum blocks

Molar Fragment Forceps Set (S-0320) Molar fragment extraction set with 3 stainless steel frag-


Molar Fragment Forceps Universal (S-0321) Universal stainless steel 4cm molar fragment forceps with19" round handle



Molar Spreader Long 3mm (S-0231) Stainless steel standard spreader with 3mm blades and 19" round handle

Molar Spreader Long 6mm (S-0238) Stainless steel standard spreader with 6mm blades and 19" round handle

Molar Forceps Universal Short (S-0230) Stainless steel forceps with flat serrated jaw and 16" flat handle

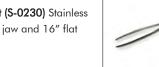



ment forceps (20, 40, and 60mm)



WWW.TERAFLOAT.COM

Version 06/2020 (US/EN/LET)